



Application Instructions

Product: Advanced Premium Wrist Brace**Model: L-43X, R-42X**

- Removable and adjustable palmar stay.
- Recommended when wrist stabilization is needed.
- Lycra lined to wick moisture away.
- Hook and loop closures for easy adjustment.
- 8" length



Order # Left	Order # Right	Size	Wrist Circum.
L - 431	R - 421	X-Small	4½" - 5½"
L - 433	R - 423	Small	5½" - 6½"
L - 435	R - 425	Medium	6½" - 7½"
L - 437	R - 427	Large	7½" - 8½"
L - 438	R - 428	X-Large	Over 8½"

SIZING:

Measure around center of wrist. Specify Right or Left.

HOW TO APPLY:

Unfasten closures. Pull brace over hand ensuring wide stay is on palm side of hand and plastic loop buckles on the top of the hand. Using free hand, thread the straps thru the plastic loops, pull until snug, and attach hook and loop. Close the top strap around the thumb. Readjust if necessary.

CARE INSTRUCTIONS:

Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L3908 (PDAC Reviewed)

Advanced Orthopaedics, Inc.

326 Main Street Harleysville, PA 19438

800.270.7074 info@advanced-orthopaedics.com