

journey SoftSecure COMMODE

PRODUCT MANUAL



COMMODE

 journey
HEALTH & LIFESTYLE®

Welcome to Journey Health & Lifestyle!

Thank you for purchasing your SoftSecure Commode. You've invested in a high-quality, premium materials commode that will facilitate comfort and safety.

Safety instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Approved for indoor use only.
- Clothing or body parts can be pinched when sitting down, standing up, inserting the toilet bucket, as well as when raising the armrests and locking them into place.
- Prevent unauthorized use, for example, by children.
- Take note of the maximum weight permitted.

Parts of the product




1. Armrest
2. Seat
3. Height adjustable leg
4. Rubber tip
5. Commode bucket

Assembly

Step 1

Attach armrests to the frame, by turning the knob and sliding the armrest into the tubes located on each side of the seat.



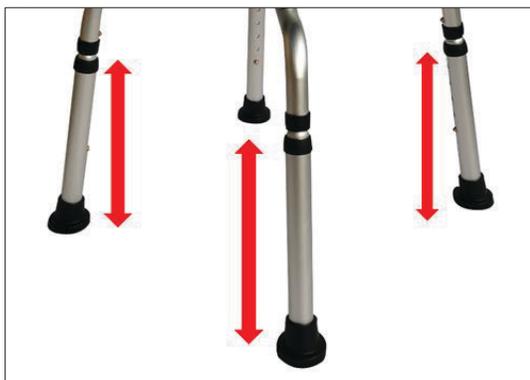

Step 2

Attach the legs to the frame by depressing push buttons into the tubes located on each side of the legs. Make sure that push buttons “snap” correctly through holes and are stable and adjusted to the same height.

Step 3

Slide Commode Bucket into guiderails located under the seat.



Product Specifications	
Product Dimension	(26"-27") x 18" x (27"-31")
Weight Capacity	300 lbs
Packaging Dimensions	29" x 19" x 9"
Net Weight	16 lbs
Material on product	Aluminum

Warranty

Journey Health & Lifestyle warrants the SoftSecure Commode frame to be free from defects in materials, workmanship and assembly for twelve (12) months from the original purchase date. The warranty does not extend to non-durable components such as the rubber tips.



 **journey**
HEALTH & LIFESTYLE®

 **journey**
SoftSecure
COMMODE

If you have any questions, call us at our toll free number:

1-800-958-8324

or visit www.shopjourney.com