

journey SoftSecure

COMMODE WITH BACKREST

PRODUCT MANUAL



COMMODE WITH BACKREST

 journey
HEALTH & LIFESTYLE®

Welcome to Journey Health & Lifestyle!

Thank you for purchasing your SoftSecure Commode with Backrest. You've invested in a high-quality, premium materials commode with backrest that will facilitate comfort and safety.

Safety instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Approved for indoor use only.
- Clothing or body parts can be pinched when sitting down, standing up, or inserting the toilet bucket.
- Prevent unauthorized use, for example, by children.
- Take note of the maximum user weight permitted.

II. PRODUCT DESCRIPTION

Parts of the product



1. Armrest
2. Seat
3. Height adjustable leg
4. Rubber tip
5. Backrest
6. Commode bucket

Assembly

Step 1

Attach armrests to the frame by turning the knobs and sliding the armrest into the tubes located on each side of the seat.



III. INSTRUCTIONS FOR USE



Step 2

Attach the legs to the frame by depressing push buttons into the tubes located on each side of the legs. Make sure that the push buttons “snap” correctly through holes and are stable and adjusted to the same height.

Step 3

Attach the backrest tube to the frame.



Step 4

Slide Commode Bucket into guiderails located under the seat.



Product Specifications	
Product Dimension	(26"-27") x 18" x (31"-35")
Weight Capacity	300 lbs
Packaging Dimensions	22" x 10" x 25"
Net Weight	20 lbs
Material on product	Aluminum

Warranty

Journey Health & Lifestyle warrants the SoftSecure Commode With Backrest frame to be free from defects in materials, workmanship and assembly for twelve (12) months from the original purchase date. The warranty does not extend to non-durable components such as the rubber tips.



 **journey**
HEALTH & LIFESTYLE®

 **journey**
SoftSecure

COMMODE WITH BACKREST

If you have any questions, call us at our toll free number:

1-800-958-8324

or visit www.shopjourney.com